

**DENTSPLY Recommended Maintenance Routines - Directions for Use can be found at [www.dentsply.co.uk](http://www.dentsply.co.uk) - Sales - Equipment Service Centre - Directions for Use**

Serial No Prefix	Date	Daily	Initial	Weekly	Initial	Three monthly	Initial
1200 Cavtron™ SPS Jet™ 30K		Sterilize handpiece and insert <b>between patients</b>		30 minute soak Sodium hypochlorite - See Section 9 DFU			
120U		Wipe with water based disinfectant <b>between patients</b> - see Section 9 DFU		followed by 30 second water rinse			
120U		Flush waterline for two minutes at start up					
120U		Remove insert & flush handpiece for 30 seconds between patients					
120U		Purge Air System for 15 seconds - see Section 9 DFU				Dismantle and inspect filters - see Section 9 DFU	
120U		Close manual Shut-off valve End of Day -prevent overnight leaking					
120U		Inspect cables for breaks and tears		Clean powder bowl cap, threads and o'ring (soft brush)			
G132 Cavtron™ Jet Plus™ 30K		Sterilize handpiece and insert <b>between patients</b>		30 minute soak Sodium hypochlorite - See Section 9 DFU			
G132		Wipe with water based disinfectant <b>between patients</b> - see Section 9 DFU		followed by 30 second water rinse			
G132		Flush waterline for two minutes at start up					
G132		Purge Air System for 15 seconds - see Section 9 DFU					
G132		Inspect cables for breaks and tears		Clean powder bowl cap, threads and o'ring (soft brush)			
G132		Close manual Shut-off valve End of Day -prevent overnight leaking					
G132		Check water line filter for Discolouration, change as required					
119U Cavtron™ SPS™ 25K		Sterilize handpiece and insert <b>between patients</b>					
119U		Wipe with water based disinfectant <b>between patients</b> - see Section 9 DFU					
119U		Ensure only 25K inserts are used on this unit					
119U with 118 Dual Select		Flush with sterile water, 2 minute at start, 30 seconds between patient					
119U with 118 Dual Select		Disinfect all surfaces End of Day -See section 10 DFU					
119U with 118 Dual Select		Flush System End of Day -See section 10 DFU		30 minute soak Sodium hypochlorite - See Section 9 DFU			
119U		Inspect cables for breaks and tears		followed by 30 second water rinse			
119U		Close manual Shut-off valve End of Day -prevent overnight leaking					
119U		Check water line filter for Discolouration, change as required					
G131 Cavtron™ SPS™ Plus™ 30K		Sterilize handpiece and insert <b>between patients</b>					
G131		Wipe with water based disinfectant <b>between patients</b> - see Section 9 DFU					
G131		Check water line filter for Discolouration, change as required					
G131		Inspect cables for breaks and tears		30 minute soak Sodium hypochlorite - See Section 9 DFU			
G131		Close manual Shut-off valve End of Day -prevent overnight leaking					
G131		Purge for two minutes Daily Start up & between patients -See section 9 DFU		followed by 30 second water rinse			
130E Cavtron™ Bobcat™ Pro 25K		Disinfect handpiece and Sterilize insert <b>between patients</b> (use barrier protection)					
130E		Wipe with water based disinfectant <b>between patients</b> - see Section 9 DFU					
130E		Ensure only 25K inserts are used on this unit					
130E		Flush waterline for two minutes at start up					
130E		Remove insert & flush handpiece for 30 seconds between patients					
130E		Disinfect all surfaces End of Day -See section 9 DFU					
130E with 118 Dual Select		Close manual Shut-off valve End of Day -prevent overnight leaking		30 minute soak Sodium hypochlorite - See Section 9 DFU			
130E		Inspect cables for breaks and tears		followed by 30 second water rinse			
130E		Check water line filter for Discolouration, change as required					
G121 Cavtron™ Select™ 25K		Disinfect handpiece and Sterilize insert <b>between patients</b> (use barrier protection)					
G121		Wipe with water based disinfectant <b>between patients</b> - see Section 10 DFU					
G121		Flush waterline for two minutes at start up				Check lavage filter inside the reservoir pump	
G121		Ensure only 25K inserts are used on this unit					
G121		Remove insert & flush handpiece for 30 seconds between patients					
G121		Disinfect all surfaces End of Day -See section 10 DFU		30 minute soak Sodium hypochlorite - See Section 9 DFU			
G121 without G123 Reservoir Pump		Close manual Shut-off valve End of Day -See section 10 DFU		followed by 30 second water rinse			
G121		Inspect cables for breaks and tears					
G121		Check lavage filter inside the reservoir pump					
G121		Close manual Shut-off valve End of Day -prevent overnight leaking					
G121		Check water line filter for Discolouration, change as required					
G124 Cavtron™ Select™ SPS™ 30K		Sterilize handpiece and insert <b>between patients</b>					
G124		Wipe with water based disinfectant <b>between patients</b> - see Section 10 DFU					
G124		Flush waterline for two minutes at start up					
G124		Remove insert & flush handpiece, time depends on Reservoir use					
G124		Disinfect all surfaces End of Day -See section 10 DFU		30 minute soak Sodium hypochlorite - See Section 9 DFU			
G124 without G123 Reservoir Pump		Close manual Shut-off valve End of Day -See section 10 DFU		followed by 30 second water rinse			
G124		Inspect cables for breaks and tears					
G124		Check lavage filter inside the reservoir pump					
G124		Close manual Shut-off valve End of Day -prevent overnight leaking					
G124		Check water line filter for Discolouration, change as required					
X Smart		Lubricate and Sterilise Contra Angle - see Section G DFU					
		Disinfect Handpiece assembly and Unit - see Section G DFU					
		Inspect cables for breaks and tears					
X Smart Dual		Lubricate and Sterilise Contra Angle - see Section K & L DFU					
		Sterilise Lip Clip and Hook - see Section K & L DFU					
		Disinfect Handpiece assembly and Unit - see Section K & L DFU					
		Inspect cables for breaks and tears					
ProMix™		Inspect cables for breaks and tears					
		Disinfect					
		Inspect casing for cracks and damage					
R RayPex 5		Inspect cables for breaks and tears					
		Disinfect measuring cable and Unit - see Section 7 DFU					
		Sterilise file clip, lip clip and Touch probe - see Section 7 DFU					
Propex II		Inspect cables for breaks and tears					
		Disinfect measuring cable and Unit					
		Sterilise file clip, lip clip					
SmartLite™		Disinfect between uses - Page 6 DFU					
		Use barrier protection - avoid composite sticking to lens					
		Battery maintenance - Page 6 DFU					
WaveOne		Inspect cables for breaks and tears					
		Disinfect Handpiece assembly and Unit - see Section 6.8 & 6.9 DFU					
		Lubricate and Sterilise Contra Angle - see Section 6.8 & 6.9 DFU					
Calamus Dual		Sterilise Pluggers and Tips - see section 6.4 DFU					
		Disinfect Handpiece assembly and Unit - see Section 6.4 DFU					
		Check Flow of Gutta Percha - see Section 6.4 DFU					
118 Dual Select		Flush with sterile water, 1 minute at start, 30 seconds between patient					
118		Disinfect all surfaces End of Day					
118		Flush System End of Day -See section 10 DFU					

